
	Formblatt	FB095	
		17.11.2016	
	Präanalytik englisch	D	1 von 2


Preanalytics: Important directions for best quality

- Carefully sway every tube several times after taking the blood
- **Do not cool or centrifuge any tube !**
- Store the tubes at room temperature and protect them from sunlight
- Please ship the tubes only in our special test kit
- Logistics:
 1. Either use our established Logistics Express Service
 2. Or ensure delivery within a maximum of 72 hours

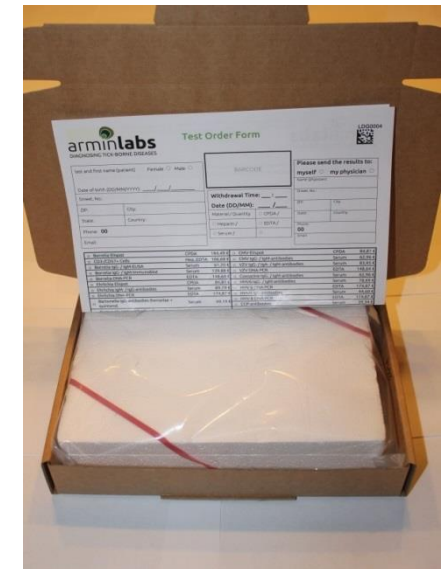

ArminLabs I Medicum Bad Aibling MVZ GmbH | CEO: Dr. med. Armin Schwarzbach, Renate
 Douwes Zirbelstraße 58, 2nd floor | 86154 Augsburg | www.arminlabs.com | info@arminlabs.com
 Amtsgericht Traunstein, HRA 27384I TAX-ID: DE321602336

Erstellt am: 17.07.2018	Geprüft am: 17.07.2018	Freigegeben am: 17.07.2018
Name: Zeidler, Sigrid	Name: Bail, Sarah	Name: Zeidler, Sigrid

	Formblatt	FB095	
	Präanalytik englisch	17.11.2016	
		D	1 von 2

Blood Drawing – Using the right tubes



Antibodies: 1 x Serum tube

Stability: Up to 14 days



EliSpot: 3 x ACD/CPDA tube

(please draw all 3 tubes if possible)

Stability: Up to 3 days



CD3-/57+ Cells: 1 x Heparin tube + 1 x EDTA tube

Stability: Up to 3 days



PCR: 1 x EDTA tube

Stability: Up to 5 days

Erstellt am: 17.07.2018	Geprüft am: 17.07.2018	Freigegeben am: 17.07.2018
Name: Zeidler, Sigrid	Name: Bail, Sarah	Name: Zeidler, Sigrid