	<b>Formblatt</b>	FB095	
		17.11.2016	
	Präanalytik englisch	D	1 von 2


### Preanalytics: Important directions for best quality

- Carefully sway every tube several times after taking the blood
- **Do not cool or centrifuge any tube !**
- Store the tubes at room temperature and protect them from sunlight
- Please ship the tubes only in our special test kit
- Logistics:
  1. Either use our established Logistics Express Service
  2. Or ensure delivery within a maximum of 72 hours



ArminLabs GmbH - CEO: Armin Schwarzbach MD PhD  
 Zirbelstraße 58, 2nd floor · 86154 Augsburg · Germany · Phone: 0049 821 780 931 50 [www.arminlabs.com](http://www.arminlabs.com)  
 Email: [info@arminlabs.com](mailto:info@arminlabs.com) · VATReg-No.: DE815543871 · Amtsgericht Augsburg HRB 29350

Erstellt am: 17.07.2018	Geprüft am: 17.07.2018	Freigegeben am: 17.07.2018
Name: Zeidler, Sigrid	Name: Bail, Sarah	Name: Zeidler, Sigrid

	<b>Formblatt</b>	FB095	
	Präanalytik englisch	17.11.2016	
		D	1 von 2

## Blood Drawing – Using the right tubes



**Antibodies: 1 x Serum tube**

Stability: Up to 14 days



**EliSpot: 3 x ACD/CPDA tube**

(please draw all 3 tubes if possible)

Stability: Up to 3 days



**CD3-/57+ Cells: 1 x Heparin tube + 1 x EDTA tube**

Stability: Up to 3 days



**PCR: 1 x EDTA tube**

Stability: Up to 5 days

Erstellt am: 17.07.2018	Geprüft am: 17.07.2018	Freigegeben am: 17.07.2018
Name: Zeidler, Sigrid	Name: Bail, Sarah	Name: Zeidler, Sigrid